




UsageGuide

**Congratulations on purchasing the
ProExtender, The #1 All Natural Penis Device!**


It is the first and best male sexual enhancement device on the market. This is a brief description of how to use the product, the most common questions and answers, the key accessories, and our overall guidelines.

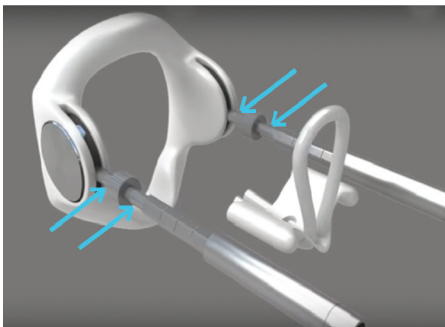

1

GET
STARTED!


Before fastening the plastic front piece, make sure there is 1mm (0.04 inch) of space on both sides of the handle on the adjustment screw, facing both the hexagonal piston, as well as the base plastic ring. This enables the adjustment screw to expand on both sides of the handle.

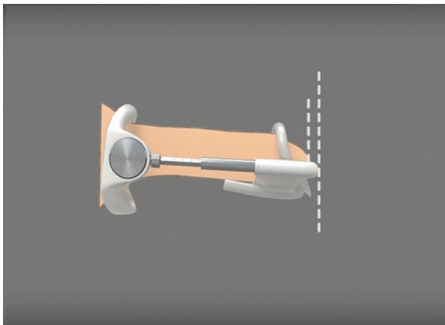

2

WHAT
IS YOUR
PERSONAL
LENGTH?


When finding the suitable number of long bars to fit you, the total length of the device should be slightly longer than your flaccid but fully stretched penis. Please add the suitable number of bars to match your current size.

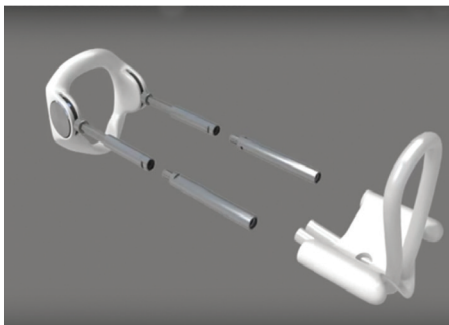

3

ATTACH THE
PLASTIC FRONT
PIECE!


Attach the plastic front piece, by inserting the bars into the holes of the front piece, pushing it towards the base firmly, bringing the springs home, and continue until you hear a click, indicating that the front is securely attached to the bars. Insert the silicone tube in the front set of holes (shown) or the comfort strap into the slots.

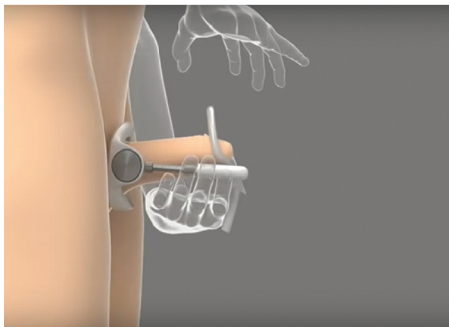

4

PUTTING
IT ON!


You should only use one of the 2 options. Try both and see which suits you best. Loosen the silicone tube (or strap) and slip the device over your penis. Push the plastic front piece towards yourself, until the springs reach home. Now tighten the silicone tube (or strap) behind the edge of your glans. Your penis should be flaccid and fully stretched. The tube should be fastened underneath the plastic front piece in the grooves. If it is a tight fit stretch the tube a bit to make it thinner before inserting into the grooves. If you use the strap simply tighten it until you reach a snug fit.

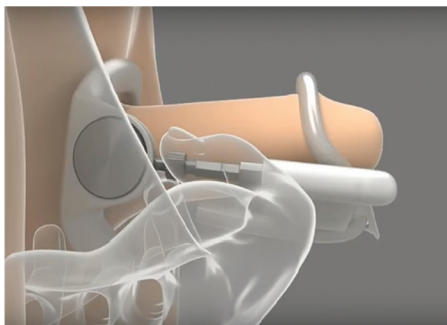

5

ADDING
MORE
TRACTION!


Release the front piece, and let the springs expand again. You should now feel a stretch of your penis (if not go back to step 2) and add more long bars). Now turn both the hand screws inwards simultaneously, and notice the hand screw expand on both sides, and the stretch increase accordingly.

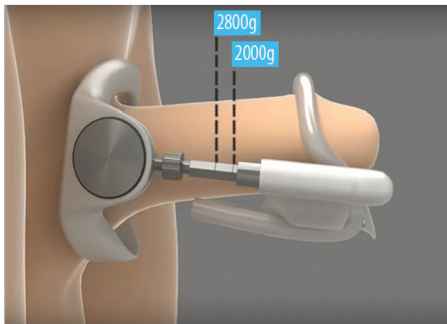

6

FINDING
YOUR
LEVEL OF
TRACTION!


As you keep turning the hand screw, the stretch becomes noticeable, and you will see the hexagonal piston submerge into the cylinder. The further into the cylinder the more traction. If you reach mark 1, the traction is 1200 grams, mark 2 is 2000 grams, and mark 3 is 2800 grams (mark 1 is not shown in the picture).

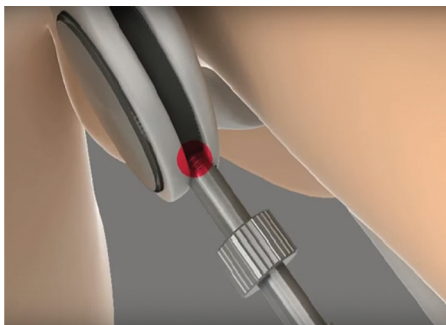

7

DON'T GO
TOO FAR!


If you continue turning the hand screw you will eventually see threads become visible, and this means it is time to stop turning. If you haven't reached the suitable amount of minimum traction (1200 grams initially), and still see the threads, you should go back to step 2 and add more long bars and retry. If the hand screw becomes undone, the warranty is void.

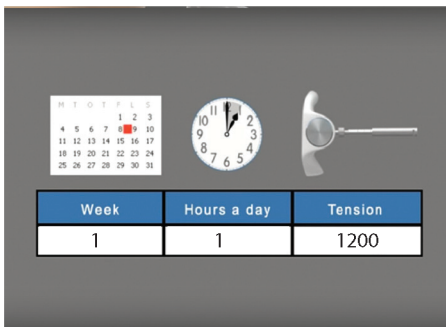

8


SLOW
START UP
ROUTINE!

Please follow a slow start up routine, and use the device for 1 hour in the first week, at 1200 g traction. Continue to add more traction as well as hours, according to our guidelines.

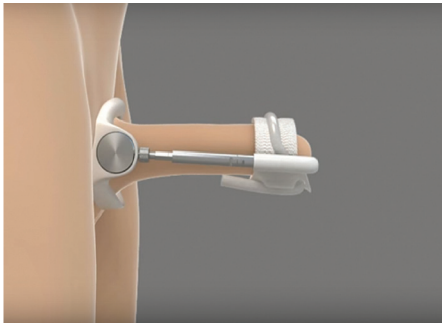


9 MEANS OF PROTECTION!

If you initially become sore/irritated behind the glans where the tube/strap touches your skin, you can use the Protection Pad as a cushion beneath the tube/strap.

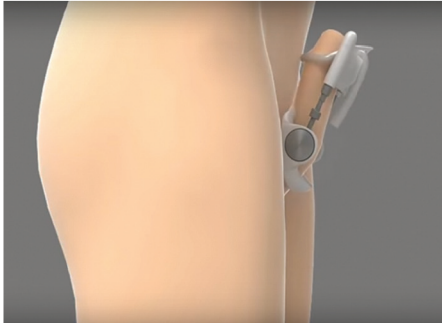

10 HOW TO WEAR!

You can wear the ProExtender in any angle suitable to you – try it out and see which is most comfortable.

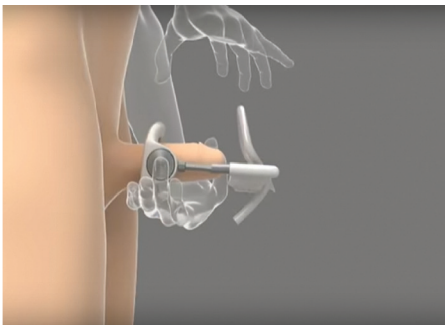

11

TAKING IT
OFF!

When you want to take off the ProExtender, simply release the tube (strap) and pull the device away from you.

WORD OF ADVICE

Please allow the process to work slowly. Results will come. This is not a sprint, but a marathon. Ongoing use will give you better results, so be patient. Don't go to extremes, and make sure you use the device only in accordance with the instructions.

PROEXTENDER. ALL YOUR QUESTIONS ANSWERED!

1. What is it, and how does it work?

It is a penis sexual enhancement device. It can boost the hardness of your erections, stimulate your partner more, and improve bedroom performance. It uses the body's natural response to traction, by creating new tissue cells. Used as your personal training device, it helps satisfy the sexual needs of yourself and your partner.

2. Does it actually work, and can you document it?

Yes! First of all, we have received an immense amount of great feedback from our clients for over 20 years. In addition, an article in "The Journal of Urology" concluded that "penile traction therapy achieved greater improvements of erectile function...intercourse satisfaction...and overall sexual satisfaction" than any other method of penis sexual enhancement.

3. How much should I wear it, and when can I expect results?

After a brief startup phase, most men wear it for 5-6 hours daily which in the first 2-3 months will provide a noticeable increase in erection quality and sexual enhancement. After using it for 4 to 6 months, the results could be double that.

4. How much improvement is possible, and are the results permanent?

In theory you will continue getting results from continued use. We are not aware of any maximum, but some very patient users have seen their outcomes continue to improve over time. Your results could indeed be permanent after stopping use, since the new cells created will still be present in any event.

5. Does this work for all men, having all sizes to begin with?

Yes – everybody should see results after correct and diligent use. The improvements mentioned are documented in clinical research on penile traction. The device will fit sizes from 4 to 22 cm (1.6 to 8.6 inches) in the stretched, flaccid state.

6. Do you guarantee that it works for me?

Yes. If you haven't achieved results after diligent, documented use, we offer a money back guarantee. Visit Proextender.com for more Information.

7. Are there any side effects?

We have no reports of adverse effects on fertility, stamina, urination or any other penile health concern.

8. Does the Proextender require experience or the help of a doctor?

No, The Proextender comes with printed and digital easy-to-understand instructions, explaining exactly how to use the device, and how it will provide sexual enhancement.

9. Do I have to follow a fixed usage pattern?

No, you can wear the device as you please. There is a start-up program for weeks 1-6, after which the results depend only on the total amount of hours worn and the traction involved.

10. Is it for men of all ages?

You must be a minimum of 18 years of age to purchase from us. As for an upper age limit, it is an individual decision – we have some users in their mid-seventies who are getting good results!

11. Is it for both circumcised and uncircumcised men?

Yes. The ProExtender can be used by both.

12. Can I have sex during the course of usage?

Yes. You can have a perfectly normal sex life during that time. As mentioned above, users indicate they have stronger erections due to the added blood flow from the traction.



3M ADVANCED COMFORT PLASTER

This plaster also works as means of protection of the skin, and to increase grip/comfort and avoid slipping. Made from a unique formula, you use this as a plaster to be placed underneath the drawn foreskin, behind the glans. Put the tube/strap on top as you would normally. When the plaster is removed, there is no pain/discomfort due to the “glue formula” – contrary to what you might imagine.



TRANSPARENT SILICONE TUBE

transparent silicone tube, one unit. Alternate method of fixation, compared to the rubber strap. Some users prefer this to the strap. Should be inserted into the holes in the front piece combo, and locked into the ridges at the bottom.



COMFORT RUBBER STRAP

Comfort strap for extra comfort if you find the silicone tubes to be uncomfortable. Some users prefer this instead of the silicone tube. Should be inserted into the slots in the front piece combo, and locked with the built-in tabs.



PROTECTION PAD

Increase grip and comfort. "Open" the protection pad with three fingers, place it at the back of your glans, and let it go. It will go back to the original size for a snug fit. Place the strap right on top of it. Can be reused.



COHESIVE GAUZE

Cohesive Gauze. Increases protection and helps secure the strap or silicone tube. Reduces slipping, and improves grip and comfort. The Cohesive Gauze should be wrapped 2-3 times around the back of the glans, and the strap should be placed right on top of it. One roll is enough for 4-6 uses.